Overview & Quick Reference Guide

1 Open Plastic Shaker Pouch*

Do NOT open foil cartridge pouch until Step 6.

2 Stick Finger

- Wash hands or clean with alcohol and dry before testing.
- Remove cap and press lancet into finger.
- Gently rotate finger to get adequate blood.

3 Collect Blood

- Gently touch blood collector to tip of blood drop to fill.
- Make sure collector is adequately filled.

4 Insert Blood Collector

- Fully insert blood collector into shaker body.
- Push hard while twisting to fully insert.

5 Mix

- Shake vigorously 6-8 times (about 5 seconds).
- This will mix the blood with the solution.
- Stand shaker on table while preparing cartridge.

6 Open Foil Cartridge Pouch*

AFTER Blood Collection

Insert cartridge into analyzer immediately and use within 2 minutes.

Tear foil pouch open at the notches on the sides.

7 Insert Cartridge

- Slide test cartridge into place.
- Analyze and test cartridge codes must match.
- If codes do not match, call Customer Service at 1-877-870-5610 (toll-free in US and Canada) or +1-317-870-5610 (direct) to walk you through the test.

8 Prepare Shaker

Remove shaker base.

9 Dispense Sample into Cartridge

- Ensure analyzer is on level surface.
- Push down completely to dispense diluted sample and remove in a continuous motion.
- Remove sampler from cartridge quickly. "OKW" will appear.

10 5 Minutes to Results

Display reads "WAIT" → "SMPL" → "67" → "0.17L"

Do not handle analyzer again until test is complete!

11 Dispose of Cartridge / Save Analyzer

- Dispose of cartridge in your household waste.
- Save analyzer for additional tests. Analyzer will display, for example, "OKW" if there is one test left and "WTE" if you just used your last test.
- Record your A1C result in the result log in Step 12.
- Analyzer will show result for 15 minutes and will turn off automatically.

12 Log Result

Chart your progress here.

Record your A1C results here, and bring this with you when you see your doctor. Mark down your A1C goal in the space provided.

| | |
|-------------|--|
| Date: | |
| A1C Result: | |
| A1C Goal: | |

| | |
|-------------|--|
| Date: | |
| A1C Result: | |
| A1C Goal: | |

| | |
|-------------|--|
| Date: | |
| A1C Result: | |
| A1C Goal: | |

| | |
|-------------|--|
| Date: | |
| A1C Result: | |
| A1C Goal: | |

Note: All instances of the box or pouches in this insert are representative ONLY. Please refer to the packaging included with your specific kit.

Note: Do NOT open foil pouch until this step. Use within 2 minutes of opening. If foil pouch is damaged, do not use.

Note: Do not handle analyzer again until test is complete!

Note: To run another test, use a new shaker, blood collector, test cartridge, and lancet from the same kit and return to Step 1.

Note: Do not take any decision of medical relevance without first consulting your healthcare practitioner. When this device is used for monitoring of an existing disease, adjust the treatment **only** if you have received the appropriate training to do so.

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